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Reusable tourniquet

### Device description

The device immobilises the blood flow in peripheral venous vessels to expand the immobilised blood vessel and enable improved vein puncture.

This product is a medical device. The device may only be used by trained medical personnel.

### Patient group

The device is intended for use on adults and children.

### Materials

Aluminium, cotton, PA, PUR

### Contraindications

The use of the device is contraindicated in patients with known sensitivities or allergies to the above materials.

The device should be used only on intact skin that is not affected by disease.

### Application

1. Place the tourniquet around the limb several centimetres proximal from the puncture site.
2. Tighten both ends of the tourniquet.
3. Push the lock to the extremity to immobilise the blood flow in the peripheral vein.
4. To loosen or release the tourniquet press the ribbed button on the lock.

### Directions

- After use, the device must be disinfected with standard chemicals.
- Dispose of the device in accordance with your internal instructions and local regulations.
- Do not use the tourniquet for bandaging of bleeding, e.g. from traumatic injuries or failed punctures.
- When placing the tourniquet be careful not to trap or injure skin or skin folds.

### Additional information

All serious incidents occurring in connection with the medical device must be reported to the manufacturer and the responsible authorities.

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