

Ref.
403.01

Bexen
MEDICAL

TRANSFER SET

CE 0318

DESCRIPTION

Transfer set with needles at both ends to transfer parenteral solutions, blood or hemoderivatives, latex-free, DEHP free, sterile and for single-use only.

CONFORMITY

Class IIa sterile product.

Oiarso, S. Coop. has implemented and maintains a quality system according to the requirements ISO 13485, ISO 9001 and ISO 14001.

CE marking according to European Directive 93/42/CEE

USE

External communication medical device in contact with the blood stream used to transfer parenteral solutions, blood or blood products between containers.

CHARACTERISTICS

Total length

70 cm

Tube

Inner diameter: 3 mm

Outer diameter: 4,1 mm

Flow regulator

Clamp

Connections

Needle protected by a cap



MATERIALS

COMPONENT	MATERIAL
Cap	Polyethylene (PE)
Needle	Stainless steel
Tube	Polyvinyl chloride PVC DEHP free, transparent
Clamp	White Polypropylene (PP)

PRESENTATION

Individual unit: Sealed peelable package

Presentation: 100 units per box

Reference	Adit.	Connection	Bag	Bottle	Unit code-EAN 13	Box code (DUN14)
403.01	No	Needle	YES	NO	8427860403017	18427860403014

STERILISATION

Process: Ethylene oxide (EO).

Expiry date: 5 years from manufacturing date.

Recommendations for use

- Use a new transfer set for each blood unit.
- Respect the asepsis rules
- Use gloves
- After the use of each and all of the elements included in the envelope, throw them away in a hazardous waste container

REUSE RISK

Do not reuse. The reuse and re-sterilization of medical devices can cause:

- Sterility problems caused by biological contamination and pyrogenic reactions.
- Cross-contamination problems caused by the use of different fluids and/or preparations.
- Physicochemical alterations that can jeopardize the device's properties and benefits.

VERIFICATION OF CONNECTIONS

Before fitting the device in its final position, and after all additional connections or disconnections, verify the stability and fastening of all the connectable elements.









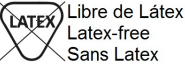







Do not use puncture needles on patients.

PERIOD OF USE

The quality of our products allows, under normal conditions of use, up to 24 hours of use. Beyond this time, we cannot guarantee that the mechanical and technical characteristics of the materials in the device will not change.

SYMBOL USED IN LABELLING

Symbol	Description	Symbol	Description
	Catalogue Number		Batch code
	Manufacturer		Date of manufacture
	Ethylene oxide sterilisation		Used-by date
	Do not reuse		Do not use if packaging is damaged
 Libre de Látex Latex-free Sans Latex	Latex free	 Sin DEHP DEHP free Sans DEHP	DEHP-free
	CE Marking		Spanish
	English		French

MANUFACTURER

Oiarso, S.Coop

Bº Zikuñaga, 57-F, Pol. Ind. Ibarluze
20120 Hernani (Guipúzcoa)

Tel : 943 33 50 20

Fax : 943 33 52 10

www.bexen.com

info@bexenmedical.com

Ref.
403.01

Bexen
MEDICAL

PRODUCT DRAWING PHOTO

